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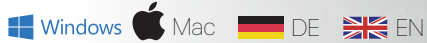
By Dr. Seeher, München, Germany

The aim must be to prepare a registration in such a way that a change in height in the articulator does not lead to a placement error in the occlusion. To do this, it must first be determined where the individual hinge axis is located in relation with the teeth. Then the position of this axis must be transferred from the patient to the articulator.

If the articulator and patient axes do not match, a registration with bite blocking inevitably leads to an occlusal error when lowering to tooth contact. It is then not possible to reproduce the contacts found on the patient in the articulator on the models. The same applies to height changes in the articulator, which are (theoretically) only possible without error if the articulator and patient axes match. This is a typical application in the reconstruction of abrasion dentitions.

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What is the software for?

This software illustrates to the user how an axis misalignment, e.g. when the articulator axis and kinematic hinge axis of the patient do not match, affects occlusion and the temporomandibular joint in mounted models. It is for understanding purposes and is not made for the use of and with patient values.

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What are the benefits?

The user is shown which change causes which effect, graphically and numerically. The following parameters can be changed: Axis error position (distance and direction), patient rotation radius, occlusal plane angle, occlusal plane inclination, anterior guidance angle, molar cusp inclination, occlusal length and the anterior thickness of the centric record. The display of various reference planes rounds off the setting options.

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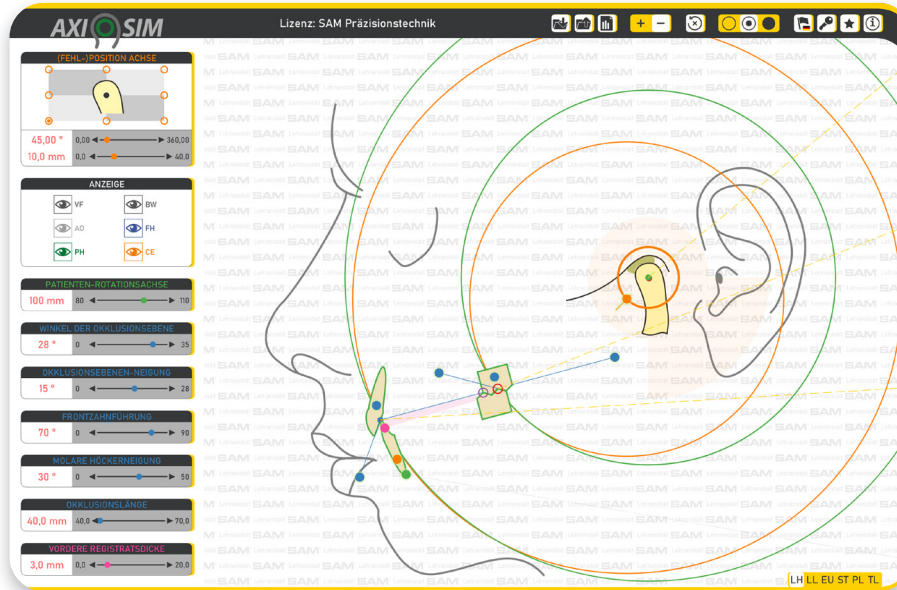
How does it work?

All parameters can be set in the full version. In the first step, you can see the dental effects, e.g. a gold crown that is too high, and in the second step, the effects of these deflections on the temporomandibular joint. Discounted versions have limited setting options.

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
- Axial placement error: 10,0mm anteriorly on the plane between patient axis and incisal contact
- Patient rotation radius: 100,0mm
- Okclusal plane inclination: 15°
- Anterior centric record thickness: 3,0mm

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
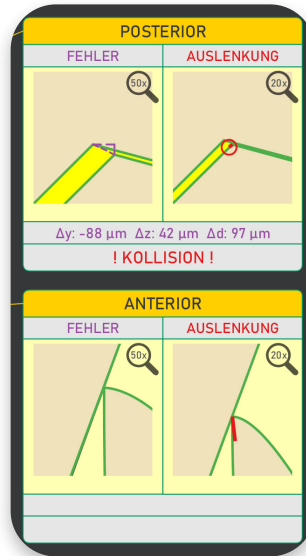
If the hold were now tested in the articulator with a shim stock foil, all teeth would hold perfectly. But beware: the slight sliding posteriorly into contact until the anterior teeth are also in contact causes an extreme deflection in the temporomandibular joint that has never been considered and calculated in this way before.

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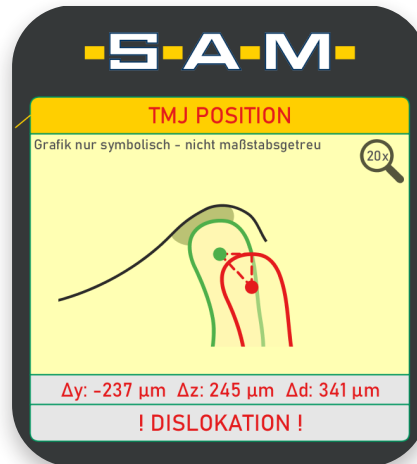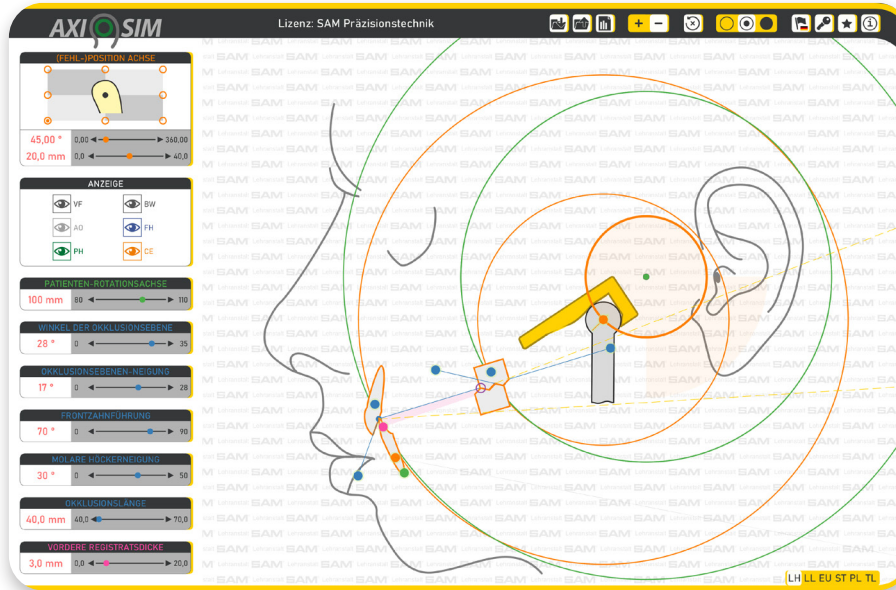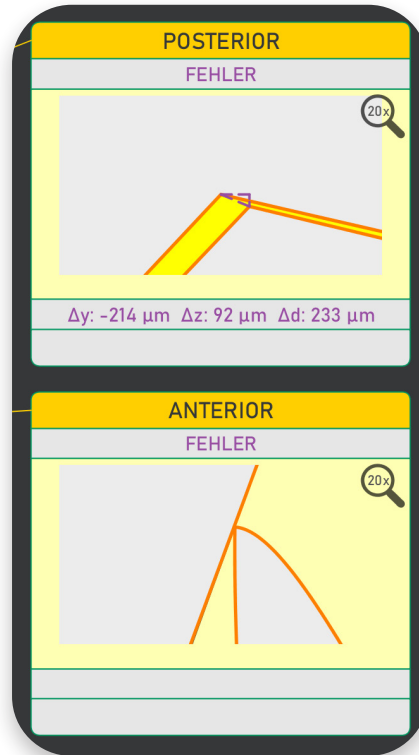
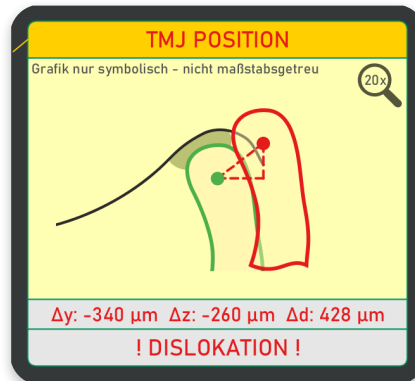
If you rotate over the (green) patient axis and close on contact, the following shear movement occurs in the temporomandibular joint.

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Conclusion: If the axis placement defect is located in extension of the occlusal plane, this can lead to massive compression of the temporomandibular joint during maximum intercuspitation.

REF AXS 100U

Professional Version
Professional version

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This version is the original version especially made for the use in universities, colleges and study clubs

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This version offers the same setting options as the version for the teaching institutions. Values can be entered more precisely and quickly, and scenarios / situations that have already been developed can be saved so that they can be shown or loaded quickly in the course. An output as pdf of the situation is also possible.

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This version is primarily intended for dentists and dental technicians who are concerned with the topic of occlusion and temporomandibular joint above average and would like to learn more about the interrelationship privately.

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This version is a special license, available only from the instructor. The owner of the license is the instructor. Any distribution to third parties is prohibited. All important functions and situations can be set for practical exercises, but the values can only be set via the scroll wheel and the situation cannot be saved.

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This version is a special license, which can be obtained only from the teaching staff. The owner of the license is the teaching institution. Any distribution to third parties is prohibited. All important functions and situations can be set for practical exercises, but the values can only be set via the scroll wheel and the situation cannot be saved.

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
This version is limited to 30 days and can only be installed once on a computer.

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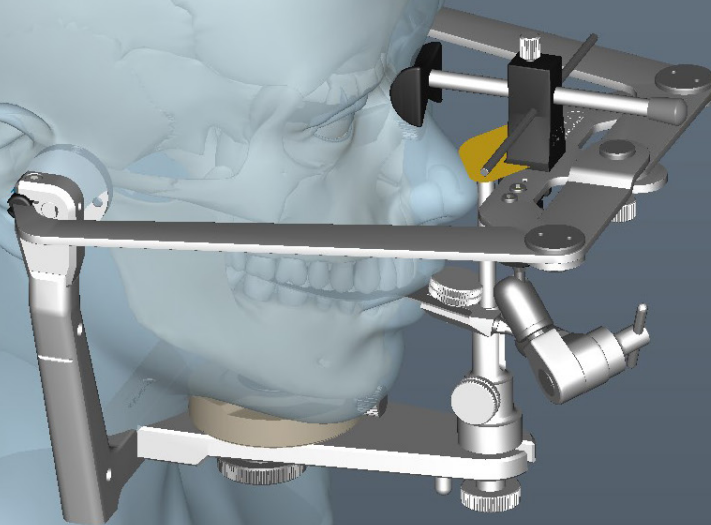Do you know our digital articulator training software Axiocomp?

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SAM® Präzisionstechnik GmbH